

New Generation **One Second Dip Card**



New Drug Screen Test



Patent design: 4 Patents


Dip into urine for 1 second, upgrading from 20 seconds

Multi-Drug Test (4 in 1)	
Barbiturates	300ng/ml
Methamphetamine	300ng/ml
Heroin	2000 ng/ml
Cannabis	25 ng/ml


New Generation **One Second Dip Card**

Convenient Procedure


1 Write ID on the dip card and remove the cap.

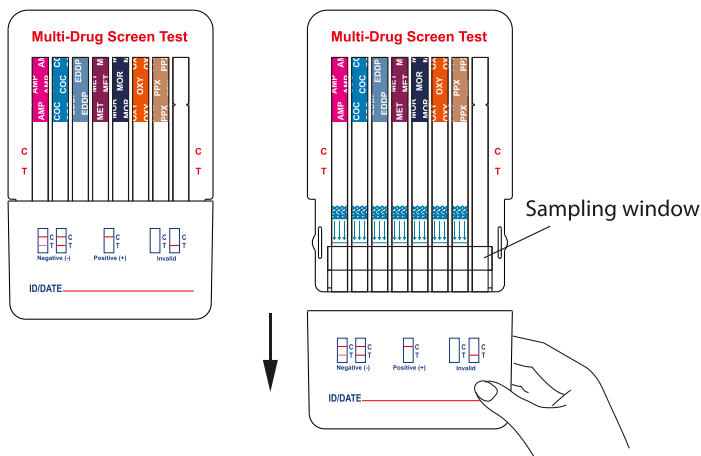


2 Immerse the dip card into the urine for minimum of 1 second.

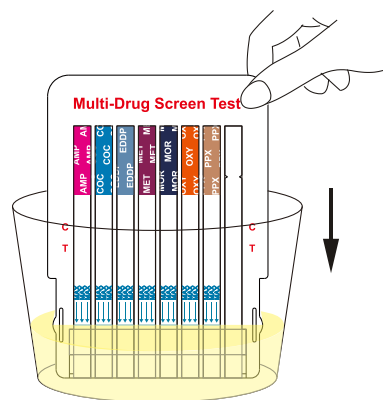

3 Negative results can be interpreted within 1 minute, Positive results should be read at 5 minutes.

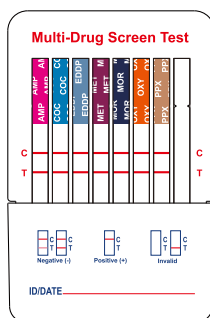

4 Interpret the results.

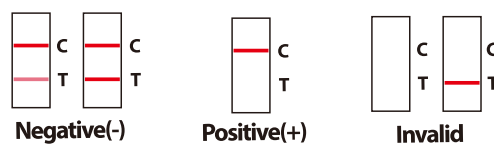

Order information

Cat.No.	Product	Specimen	Format	Pack
DMDR-DP14	Multi-Drug 4 Drugs Rapid Test Dip Card	Urine	Dip Card	25T